



## 7 Product Specific Instructions – Power Wheelchairs

### – B400 – (versions with standard seat and contour seat)

For assembly of the 491S00=SK024 fixation kit you will need the following tools: Ring wrenches sizes 13 and 17, Allen key size 6

#### Attaching the anchoring points to the product (to be carried out by Otto Bock only)



Screw the supplied pins into the bores on the seat frame on the left and right.



The pin and the front eyelet must be ordered in the ISO set.



The rear belt eyelets are already installed on the drive unit sustainer at the factory.


#### Routing the lap belt



B400 showing the correct positioning of the tie-down straps.



Pass the two ends of the lap belt towards the back from the sides of the seat and attach each end of the lap belt to the pin on the respective opposite side.

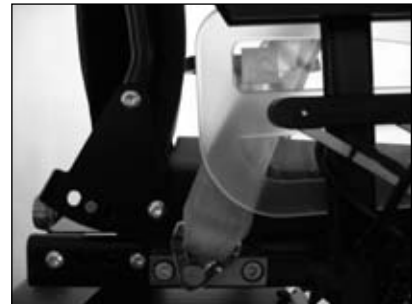

This power wheelchair is equipped with an angle-adjustable back rest. Prior to operation, adjust the back rest to a nearly upright position (max. 6°) and check if the locking mechanism has engaged on both sides. Observe the specifications in the instructions for use of the power wheelchair.



This power wheelchair may be equipped with a choice of electrical or mechanical tilt feature. The tilt must be adjusted to 0° prior to operation.